LymphoSign

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GENEXPATH LymphoSign user guide

User precautions.

In vitro diagnostic medical device according to Directive (EU) 98/79/EC

For *in vitro* diagnostic use

It is for professional use only.

Read all information in this user guide before use.

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contact@genexpath.com

support@genexpath.com

| | |
|--|----|
| ENGLISH..... | 2 |
| Important precautions. | 5 |
| General recommendations. | 5 |
| Icons | 5 |
| Intended use..... | 5 |
| Test principle..... | 6 |
| Reagents..... | 7 |
| Content of GENEXPATH LymphoSign reagent kit. | 7 |
| Format of reagent kits sold and quantities:..... | 8 |
| Reagents not supplied in the reagent kit:..... | 8 |
| Required equipment:..... | 9 |
| Before starting..... | 9 |
| Biological samples..... | 9 |
| Solutions to prepare. | 9 |
| Programming the thermocyclers. | 10 |
| Programme 1: Pre-PCR..... | 10 |
| Programme 2: PCR. | 11 |
| Detailed protocol..... | 11 |
| Step 1: Reverse transcription..... | 11 |
| Required reagents. | 11 |
| Step 1.a: Reverse transcription; denaturation of RNA and hybridization of random hexamers..... | 11 |
| Step 1.b: Reverse transcription; conversion of RNA to cDNA..... | 12 |
| Step 2: Hybridization of probes. | 12 |
| Required reagents. | 12 |
| Hybridization of probes..... | 12 |
| Step 3: Ligation..... | 13 |
| Required reagents. | 13 |
| Ligation..... | 13 |
| Step 4: Amplification and incorporation of barcodes and adapters. | 14 |
| Required reagents. | 14 |
| Amplification. | 14 |
| Step 5: Purification and assay of sequencing libraries. | 15 |
| Required reagents. | 15 |
| Step 5.a: Purification of sequencing libraries. | 15 |

| | |
|--|----|
| Step 5.b: Assay of sequencing libraries:..... | 15 |
| Step 6: Dilution, pooling and sequencing of libraries. | 15 |
| Required reagents. | 15 |
| Sequencing on an Illumina MiSeq sequencer. | 16 |
| • Step 6.a: Dilution and pooling of libraries..... | 16 |
| • Step 6.b: Denaturation and dilution of the libraries pool..... | 16 |
| • Step 6.c: Preparation of the sequencing primer. | 16 |
| • Step 6.d: Preparation of the injection sheet. | 16 |
| • Step 6.e: Start of sequencing. | 17 |
| Sequencing on an Illumina NextSeq 500/550 platform. | 17 |
| • Step 6.a: Dilution and pooling of libraries..... | 17 |
| • Step 6.b: Denaturation and dilution of the libraries pool..... | 17 |
| • Step 6.c: Preparation of the sequencing primer. | 17 |
| • Step 6.d: Preparation of the injection sheet. | 18 |
| • Step 6.e: Start of sequencing. | 18 |
| Step 7: Results analysis. | 18 |
| Limits of the procedure | 19 |
| Analytical performance | 19 |
| Repeatability | 19 |
| Interoperability | 20 |
| Reproducibility | 20 |
| Bibliography..... | 21 |
| Symbol table..... | 21 |
| Notes | 21 |

Important precautions.

General recommendations.

- Usable for *in vitro* diagnostic use
- Follow laboratory best practices in terms of handling PCR products (wear disposable overalls and gloves, mark out dedicated zones for pre- and post-PCR, use filter tips).
- Also take precautions to avoid nuclease contamination likely to cause RNA and DNA degradation (use nuclease-free reagents and consumables).
- Ensure that the thermocyclers are in working order and calibrated based on manufacturer recommendations.
- It is particularly important not to substitute reagents not included in the kit, particularly buffers and enzymes used during the reverse transcription, ligation and PCR amplification steps. The incubation temperatures and times, as well as volumes and concentrations, must also be respected.
- **GENEXPATH LymphoSign** reagents are only intended for use with Illumina's MiSeq or NextSeq 500/550 sequencing platforms.
- Safety data sheets are available in the user space.
- If the user lights out errors in the instruction manual: please send an email at contact@genexpath.com.
- Any serious incident occurring in connection with the device must be notified to us at contact@genexpath.com.

Icons

Important points and critical steps of the protocol that could compromise result quality.

Steps where the protocol may be suspended.

Intended use.


This protocol is intended for **GENEXPATH LymphoSign** testing. It is used to prepare sequencing libraries for Illumina's MiSeq or NextSeq 500/550 sequencers.

The fastQ files generated using this test contain data about the expression levels of over 130 genes and genetic markers. They are analysed using the **GENEXPATH RT-MIS** platform, which contains a specific sequence demultiplexing application, as well as an Artificial Intelligence

algorithm which compares expression profiles obtained with those for the main types of Non-Hodgkin lymphoma.


Test principle.

The **GENEXPATH LymphoSign** test uses a ligation-dependent RT-PCR method (LD-RT-PCR). This semi-quantitative technique helps simultaneously assess the expression levels of a large number of genetic markers such as genes, somatic mutations or chromosome translocations using oligonucleotide probe pairs specific to each of these markers.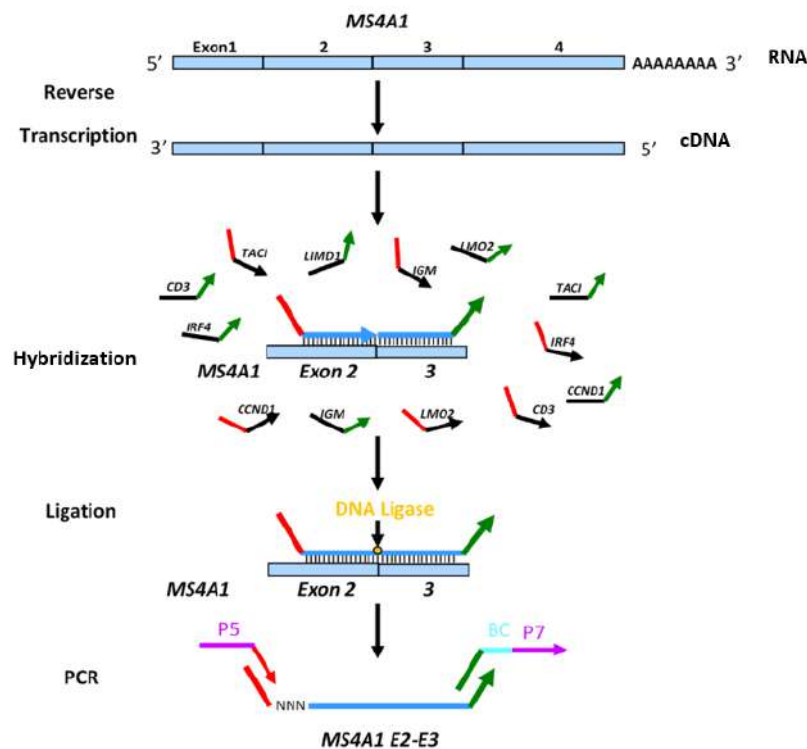

Four steps are sufficient to obtain libraries from a total RNA extraction.

- A reverse transcription (RT) step.
- A hybridization of specific oligonucleotide probes step.
- A ligation step.
- A PCR amplification step.

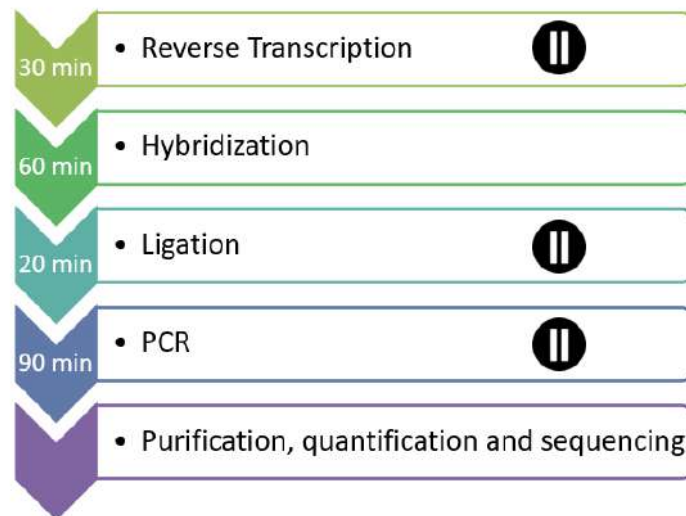No purification is required until the libraries are obtained, which limits material losses and ensures this technique has excellent sensitivity. Also, the gene sequences targeted by the probes are particularly short (between 40 and 60 bases), which ensures excellent robustness with regards to RNA degradation.

LD-RT-PCR is therefore a particularly appropriate approach for analysing difficult biological samples like fixed paraffin-embedded tissue biopsies.

For each sample, around 10^5 sequences are sufficient to obtain an analysable expression profile, which helps test a large number of samples simultaneously in the same sequencing FlowCell. To optimise costs, **GENEXPATH LymphoSign** libraries can also be loaded at the same time as other sequencing libraries, generated by other methods.

Reagents.

Content of GENEXPATH LymphoSign reagent kit.

| | |
|---|------------|
| GENEXPATH LymphoSign probe mix | GEP-LSPM |
| GENEXPATH LymphoSign barcodes | GEP-BC-xxx |
| GENEXPATH LymphoSign sequencing primer | GEP-SP-001 |
| XXX: barcode number | |

On receipt, these reagents should be stored between -25°C and -15°C .

They are ready to use and do not need to be diluted.

The shelf life of the reagents is 1 year.

Return to storage conditions immediately after use.

Do not use reagents after their expiration date stated on the label.

Format of reagent kits sold and quantities:

| | Reagent kit - U = number of analyses | | | |
|---|--------------------------------------|--------------|--------------|--------------|
| | 8U | 16U | 24U | 48U |
| GEP-LSPM probe mix | 30 µL | 48 µl | 54 µl | 108 µL |
| Barcodes GEP-BC-xxx (from 001 to 032 depending on number of analyses purchased) BC=barcode | 8 BC | 8 BC | 12 BC | 24 BC |
| | N°001 to 008 | N°017 to 024 | N°001 to 012 | N°009 to 032 |
| | 5µL/BC | 9µL/BC | 9µL/BC | 9µL/BC |
| GEP-SP-001 sequencing primers | 60 µL | 96 µL | 144 µL | 288 µL |

Reagents are supplied in larger quantities than actually required. After the ordered number of analyses has been completed, they should be discarded. If a new order is placed, new reagents will be delivered.

For a reagent kit with more than 8 analyses, each barcode will be used for 2 different analyses.

Reagents not supplied in the reagent kit:

| Reagents | Suppliers and references |
|--|---------------------------------|
| Invitrogen™ M-MLV Reverse Transcriptase (200U/µL) | Fisher Scientific, ref 10338842 |
| Invitrogen™ dNTPs set (100mM) | Fisher Scientific, ref 10083252 |
| Invitrogen™ Random Primers | Fisher Scientific, ref 10646313 |
| SALSA MLPA Buffer (180 µL) | MRC Holland, ref SMR33 |
| SALSA Ligase Buffer A (360 µL) | MRC Holland, ref SMR12 |
| SALSA Ligase Buffer B (360 µL) | MRC Holland, ref SMR13 |
| SALSA Ligase-65 (115 µL) | MRC Holland, ref SMR20 |
| Q5® Hot Start High-Fidelity 2X Master Mix | New England BioLabs, ref M0494 |
| AMPure XP (Magnetic beads) | Beckman Coulter, ref A63880 |
| Qubit® dsDNA HS Assay Kit | Fisher Scientific, ref 10616763 |
| Sequencing reagents | Illumina |
| TE buffer (10 mM Tris-Acetate pH 8.0, 1 mM EDTA) | Variable |

| | |
|-------------------------|----------|
| Ethanol 100% | Variable |
| NaOH 1 N | Variable |
| Tris Buffer 200 mM pH 7 | Variable |
| Nuclease-Free Water | Variable |

On receipt and between each use, these reagents should be stored based on supplier recommendations.

Required equipment:

| Equipment | Suppliers and references |
|---|--------------------------------------|
| Thermocycler in pre-PCR zone | Variable |
| Thermocycler in post-PCR zone | Variable |
| Qubit® Fluorometer (or equivalent) | Thermo Fisher Scientific, ref Q33238 |
| Qubit® Assay Tubes | Fisher Scientific, ref 12037609 |
| DynaMag™-96 Side Magnet - Magnetic plate (AMPure XP purification) | Thermo Fisher Scientific, ref 12331D |
| PCR 200 µL plates and tubes | Variable |

Before starting.

Biological samples.

The **GENEXPATH LymphoSign** test is used to prepare sequencing libraries from total RNA extractions of tumour biopsies or human cell lines. This test is only applicable to Non-Hodgkin lymphomas.

These samples may be fresh, frozen or formalin-fixed, paraffin-embedded (FFPE).

To extract RNA from fixed tissues, we recommend using the kit Promega Maxwell® RSC RNA FFPE (Promega, ref AS1440 and AS4500).

The amount of RNA to be analysed should be between **50 and 500 ng, in a volume of 2 µL**. If the concentration of the solutions to be analysed is too high, this RNA can be diluted in nuclease-free water.

Solutions to prepare.

- Random hexamers (Random Primers, Fisher Scientific, ref 10646313).
Working solution concentration: 100 µM
- Dilute 100 µL of the initial solution (at 3 µg/µL) by adding 1487 µL of nuclease-free water.

- Aliquot and store between -25°C and -15°C.
- dNTPs (dNTPs set, Fisher Scientific, ref 10083252).
Working solution concentration: 10 mM
- Mix the 4 stock solutions (250 µL each) and dilute by adding 1.5 ml of nuclease-free water.
- Aliquot and store between -25°C and -15°C.

Programming the thermocyclers.

To limit the risks of contamination, use two thermocyclers, one in the pre-PCR zone and one in the post-PCR zone.

Two programmes are required:

- The first is for the first three steps of the protocol: **reverse transcription of RNA to cDNA, hybridization of oligonucleotide probes, and ligation**. It must be run on the thermocycler located in the pre-PCR zone.
- The second is used to **amplify ligation products and incorporate the barcodes and adapters required for sequencing**. It must be run on the thermocycler located in the post-PCR zone.

Programme 1: Pre-PCR.

As the reaction volumes are small, ensure that the temperature of the thermocycler's heated lid remains high (95°C) at all steps of the programme to avoid evaporation.

Breaks at 4°C are provided between the different programme steps to add the required reagents.

Step 1: Reverse transcription.

Step 1a: Denaturation of RNA and hybridization of hexamers.

- Heated lid: 95°C
- 2 Minutes 80°C
- 5 Minutes 37°C
- 4°C infinite

Step 1b: Reverse transcription of RNA to cDNA.

- Heated lid: 95°C
- 15 Minutes 37°C
- 2 Minutes 98°C
- 4°C infinite

Step 2: Hybridization of probes.

- Heated lid: 95°C
- 2 Minutes 95°C

- 60°C infinite (1 hour of hybridization)

Step 3: Ligation.

- Heated lid: 95°C
- 54°C infinite (distribution of ligation mix)
- 15 minutes 54°C
- 5 minutes 98°C
- 4°C infinite

Programme 2: PCR.

- Heated lid: 95°C
- 6 minutes 94°C
- 35 x (30 seconds 94°C; 30 seconds 58°C; 30 seconds 72°C)
- 4 minutes 72°C
- 4°C infinite

Detailed protocol.

Step 1: Reverse transcription.

This step must be completed in the pre-PCR zone.

Required reagents.

- dNTPs 10 mM, random hexamers 100 µM, M-MLV RT kit (Reverse transcription buffer 5 x, DTT 100 mM, M-MLV RT enzyme), total RNA extraction to test (25 to 250 ng/µL).

It is recommended to carry out the entire procedure in 200 µL PCR plates or tubes.

Step 1.a: Reverse transcription; denaturation of RNA and hybridization of random hexamers.

- Thaw the following reagents, then keep them on ice or in a cooling rack:
 - Reverse transcription buffer 5 x
 - DTT 100 mM
 - dNTPs 10 mM
 - Hexamers 100 µM
- Prepare a reverse transcription mix. For each sample, mix (for a total volume of 3.75 µL per reaction):
 - Reverse transcription buffer 1.25 µL
 - DTT 0.5 µL
 - dNTPs 1 µL
 - Hexamers 1 µL

- Distribute this mix in 200 µL PCR tubes (3.75 µL per tube) kept on ice or in a cooling rack:
- Add 2 µL of each of the total RNA solutions (50 to 500 ng) to the different tubes.
- Vortex, centrifuge briefly.
- Place the tubes in the thermocycler in the pre-PCR zone, and proceed to **step 1.a of the Pre-PCR programme** (denaturation and hybridization of hexamers).

Step 1.b: Reverse transcription; conversion of RNA to cDNA.

- At the end of step 1a, when the thermocycler temperature has dropped to 4°C, briefly centrifuge the tubes, then place them on ice or in a cooling rack.
- Add 0.5 µL of reverse transcriptase (M-MLV RT) to each tube.
- Centrifuge briefly.
- Place the tubes back in the thermocycler.
- Check the temperature of the heated lid (95°C).
- Proceed to **step 1.b of the pre-PCR programme** (reverse transcription of RNA to cDNA).

Then proceed directly to step 2, or keep the ligation products between -25°C and -15°C.

Step 2: Hybridization of probes.

This step must be completed in the pre-PCR zone.

Required reagents.

- **GENEXPATH LymphoSign** probe mix (GEP-LSPM), SALSA MLPA Buffer.

Hybridization of probes.

- **At the end of step 1b**, when the thermocycler temperature has dropped to 4°C, remove the tubes, centrifuge them briefly, and place them on ice or in a cooling rack.
- Thaw the Salsa MLPA buffer and the **GENEXPATH LymphoSign** probe mix, then keep them on ice or in a cooling rack.
- Prepare a hybridization mix. For each sample, mix (for a total volume of 3 µL per reaction):
 - Salsa MLPA Buffer 1.5 µL
 - **GENEXPATH LymphoSign** probe mix 1.5 µL

- Vortex, centrifuge briefly.
- Add 3 μ L of this mix to each cDNA tube.
- Centrifuge briefly.
- Place the tubes back in the thermocycler.
- Check the temperature of the heated lid (95°C).
- Proceed to **step 2 of the pre-PCR programme** (hybridization of probes).

Step 3: Ligation.

This step must be completed in the pre-PCR zone.

Required reagents.

SALSA Ligase Buffer A, SALSA Ligase Buffer B, SALSA Ligase 65, nuclease-free water.

Ligation.

- 15 minutes before the end of step **2**, thaw SALSA Ligase Buffer A and SALSA Ligase Buffer B and keep them on ice or in a cooling rack.
- Place the Salsa Ligase 65 enzyme on ice or in a cooling rack.
- Prepare a ligation mix. For each sample, mix (for a total volume of 32 μ L per reaction):
 - o Nuclease-free water 25 μ L
 - o Salsa Ligase Buffer A 3 μ L
 - o Salsa Ligase Buffer B 3 μ L
- Vortex, centrifuge briefly
 - o Salsa Ligase 65 1 μ L
- Vortex, centrifuge briefly.
- After 60 minutes of incubation, proceed to **step 3 of the pre-PCR programme (ligation)**.
- Lower the temperature of the heated block to 54°C.
- Add 32 μ L of the ligation mix directly to each tube, without removing them from the heated block.
- After distributing the mix, proceed to the next step of the programme (15 minutes at 54°C, 5 minutes at 98°C).

At the end of this step, when the temperature of the PCR block drops to 4°C, immediately proceed to step 4 (PCR amplification) or freeze the ligation products (between -25°C and -15°C).

After this step, do not keep products at higher temperatures (e.g. 4°C or room temperature) to avoid non-specific ligations which could result from residual enzyme activity.

Step 4: Amplification and incorporation of barcodes and adapters.

In this step, the ligation products are amplified by PCR thanks to additional tails at the end of the probes. These amplifications are carried out using pairs of primers supplied in the **GENEXPATH LymphoSign** barcode tubes (GEP-BC-xxx).

To allow analysis of several samples in the same FlowCell, the PCR 3' primer has a molecular barcode which will be recognised by the demultiplexing algorithm of the **GENEXPATH RT-MIS** platform.

Required reagents.

GENEXPATH LymphoSign barcodes (GEP-BC-xxx), Q5® Hot Start High-Fidelity 2X Master Mix, nuclease-free water.

Amplification.

- Prepare an amplification mix in the pre-PCR zone. For each sample, mix (for a total volume of 18 µL per reaction):
 - o Q5® High-Fidelity 2X Master Mix 12.5 µL
 - o Nuclease-free water 5.5 µL
- Vortex, centrifuge briefly.
- Distribute 18 µL of this amplification mix in the different wells in a PCR plate.
- Add 5 µL of ligation products generated in step 3 to each of the wells.
- Add 2 µL of **GENEXPATH LymphoSign** barcode to each well.

Use different BEP-BC-xxx barcodes for each tested sample.

- Place the plate in the thermocycler in the **post-PCR zone**.
- Start **programme 2** (PCR).

At the end of the programme, when the temperature of the thermocycler drops to 4°C, quickly proceed to step 5 (purification) or freeze the amplification products between -25°C and -15°C.

Do not keep these products at higher temperatures for a long period (e.g. 4°C in the thermocycler or at room temperature).

Step 5: Purification and assay of sequencing libraries.

At the end of the amplification step, the sequencing libraries should be purified to eliminate PCR primers and unincorporated nucleotides. This purification uses AMPure XP magnetic beads. The libraries should be assayed via fluorimetry before loading in the sequencer.

Required reagents.

Ethanol 100%, nuclease-free water, AMPure XP beads, TE buffer (10 mM Tris-Acetate pH 8.0, 1 mM EDTA), Qubit® dsDNA HS Assay.

Step 5.a: Purification of sequencing libraries.

Ensure that the beads are completely re-suspended before use.

- Purify 25 µL of PCR products with 45 µL of AMPure XP beads (following manufacturer recommendations).
- Elute the purified PCR products in 50 µL of TE buffer.

After purification, the libraries can be stored between -25°C and -15°C before sequencing.

Step 5.b: Assay of sequencing libraries:

- Assay 10 µL of each sequencing library via fluorimetry (Qubit dsDNA HS Assay kit, as per supplier recommendations).

Step 6: Dilution, pooling and sequencing of libraries.

After purification, the **GENEXPATH LymphoSign** libraries must be diluted, pooled and loaded in the sequencer.

For optimal results, a minimum of 10⁵ sequences should be read for each sample.

Unlike most classic sequencing libraries, the reading of molecular barcodes required for demultiplexing **GENEXPATH LymphoSign** sequences takes place during read1. These sequences are therefore not demultiplexed automatically by the sequencer, and will be saved in “Undetermined” fastQ files. Demultiplexing is carried out using the specific algorithm provided on the **GENEXPATH RT-MIS** platform.

Required reagents.

GENEXPATH LymphoSign sequencing primer (GEP-SP-001), Illumina sequencing reagents.

Sequencing on an Illumina MiSeq sequencer.

For detailed information about dilution and denaturation of libraries, preparation of the sequencing primer, the injection sheet and the start of sequencing, refer to the Illumina guide to the MiSeq system.

- ***Step 6.a: Dilution and pooling of libraries.***

- Dilute each **GENEXPATH LymphoSign** library at a concentration between 2 nM and 4 nM, considering an average amplified fragment size of 150 pb.
- Pool **GENEXPATH LymphoSign** libraries in the equivalent volume.
- If other libraries are sequenced on the same FlowCell, adjust the concentrations of different pools, then combine them to obtain the desired sequence numbers (minimum 10^5 sequences for each **GENEXPATH LymphoSign** library).

Example: For a pool of 10 **GENEXPATH LymphoSign** libraries requiring 1 M sequences (10^5 sequences for each library), sequenced with a pool of libraries B at the same concentration and requiring 3 M sequences, pool 1 μL of the **GENEXPATH LymphoSign** libraries pool and 3 μL of the pool of libraries B.

- ***Step 6.b: Denaturation and dilution of the libraries pool.***

- Denature and dilute the final pool based on recommendations in the Illumina guide to the MiSeq system, to obtain a final load concentration of 8 to 10 pM.

- ***Step 6.c: Preparation of the sequencing primer.***

- If the **GENEXPATH LymphoSign** libraries pool is sequenced alone, add 3 μL of the **GENEXPATH LymphoSign** sequencing primer (GEP-SP-001) to 597 μL of HT1 buffer, then place this 600 μL in well 18 of the MiSeq reagent cartridge.
- If the **GENEXPATH LymphoSign** libraries pool is loaded with other libraries sequenced using Illumina sequencing primers, pipette the entire content of well 12 (around 600 μL), add 3 μL of the **GENEXPATH LymphoSign** sequencing primer, then place this mix in well 18 of the cartridge.

- ***Step 6.d: Preparation of the injection sheet.***

- If the **GENEXPATH LymphoSign** library is sequenced alone, create the injection sheet to generate FASTQ files with 120 cycles in read 1.
- If **GENEXPATH LymphoSign** libraries are combined with other sequencing libraries, generate the injection sheet using the usual parameters, without entering the **GENEXPATH LymphoSign** samples.

- Specify the use of custom during run setup (with Local Run Manager on the Create Run page. In manual run mode, on the Run Setup screen).

In all cases, ensure that read 1 is carried out with a minimum of 120 cycles and the use of custom sequencing primer is specified.

- In all cases, the **GENEXPATH LymphoSign** library sequences will not be demultiplexed by the sequencer, but will be saved in an “Undetermined” FastQ file, which will then be loaded on the **GENEXPATH RT-MIS** platform.
- **Step 6.e: Start of sequencing.**
 - Start sequencing by following the procedure described in the Illumina guide to the MiSeq system.

Sequencing on an Illumina NextSeq 500/550 platform.

For detailed information about dilution and denaturation of libraries, preparation of the sequencing primer, the injection sheet and the start of sequencing, refer to the Illumina guide to the NextSeq system.

- **Step 6.a: Dilution and pooling of libraries.**
 - Dilute each **GENEXPATH LymphoSign** library at a concentration between 0.5 nM and 4 nM, considering an average amplified fragment size of 150 pb.
 - Pool **GENEXPATH LymphoSign** libraries in the equivalent volume.
 - If other libraries are sequenced on the same FlowCell, adjust the concentrations of different pools, then combine them to obtain the desired sequence numbers (minimum 10^5 sequences for each **GENEXPATH LymphoSign** library).

Example: For a pool of 10 **GENEXPATH LymphoSign** libraries requiring 1 M sequences (10^5 sequences for each library), sequenced with a pool of libraries B at the same concentration and requiring 3 M sequences, pool 1 μ L of the **GENEXPATH LymphoSign** libraries pool and 3 μ L of the pool of libraries B.

- **Step 6.b: Denaturation and dilution of the libraries pool.**
 - Denature and dilute the final pool based on recommendations in the Illumina guide to the NextSeq system, to obtain a final load concentration of 0.8 pM to 1 pM.
- **Step 6.c: Preparation of the sequencing primer.**
 - If the **GENEXPATH LymphoSign** libraries pool is sequenced alone, dilute 6 μ L of the **GENEXPATH LymphoSign** sequencing primer in 1994 μ L of HT1 buffer, then place this 2 mL in well 7 of the NextSeq reagent cartridge.

- If the **GENEXPATH LymphoSign** libraries pool is combined with other libraries sequenced using Illumina sequencing primers, pipette the entire content of well 20 (around 2 mL), add 6 µL of the GEP-SP sequencing primer, then place this mix in well 7 of the cartridge.
- **Step 6.d: Preparation of the injection sheet.**
 - If the **GENEXPATH LymphoSign** library is sequenced alone, create the injection sheet to generate FASTQ files with 120 cycles in read 1.
 - If **GENEXPATH LymphoSign** libraries are combined with other sequencing libraries, generate the injection sheet using the usual parameters, without entering the **GENEXPATH LymphoSign** samples.
 - Specify the use of custom during run setup (with Local Run Manager on the Create Run page. In manual run mode, on the Run Setup screen).

In all cases, ensure that read 1 is carried out with a minimum of 120 cycles and the use of custom sequencing primer is specified.

- In all cases, the **GENEXPATH LymphoSign** library sequences will not be demultiplexed by the sequencer, but will be saved in the four “Undetermined” FastQ file, which will then be loaded on the **GENEXPATH RT-MIS** platform.
- **Step 6.e: Start of sequencing.**
 - Start sequencing by following the procedure described in the Illumina guide to the NextSeq system.

Step 7: Results analysis.

The sequence files generated by the Illumina sequencing platform (MiSeq or NextSeq) in FastQ format must be analysed using the **GENEXPATH RT-MIS** software available via the customer space at the following address: <https://connect.genexpath.com/>.

To help download the FastQ file, it should not be decompressed (fastq.gz).

This software is a comprehensive bioinformatics solution which includes different data processing algorithms. It carries out demultiplexing to assign sequences to each sample. It then precisely identifies the gene expression markers and quantifies them.

The **GENEXPATH LymphoSign** test is based both on quantification of quantitative markers (gene expression) but also qualitative markers (presence or absence of mutation and chromosome translocations).

GENEXPATH RT-MIS includes an artificial intelligence algorithm to help the user to use its results. It generates concise and transparent reports ranging from implementation of sequencing reactions to automated analysis of sequencing results.

GENEXPATH RT-MIS requires sequencing files to be loaded in FASTQ format, as well as the list of barcodes used during testing.

GENEXPATH RT-MIS assesses the sequencing quality of each sample by quantifying the number of reads identified and the number of UMI (unique molecular identifiers) detected. For each sample, **GENEXPATH RT-MIS** generates a graph to help the user to analyse the data. The identification data for each marker and for each sample is available to download.

GENEXPATH RT-MIS includes a user guide directly accessible online to explain how to use the tool, to describe all generated results and explain the presentation of results.

The company **GENEXPATH** does not permanently store results generated by the software **GENEXPATH RT-MIS**. The data must be downloaded directly after each analysis and saved by the user in their document management system.

Limits of the procedure

- The LymphoSign test allows the simultaneous evaluation of the expression levels of a large number of genetic markers such as genes, somatic mutations or chromosomal translocations using specific probes. It is intended for the diagnosis of non-Hodgkin's lymphoma. Samples tested should be FFPE or frozen tissue biopsies.
- The performance demonstrated in the “Analytical performance” paragraph has been validated according to the instructions set out above.
- A low amount of RNA or a low quality sample may cause an uninterpretable result.
- Sequencing must be performed with Illumina technology sequencers (Miseq and NextSeq).

Analytical performance

Repeatability

The repeatability of the LymphoSign test is defined as its ability to accurately quantify each of the test markers. The same sample analyzed in triplicate by the LymphoSign signature was studied. For each marker, the dispersions between the measured values and the expected mean values are reported on the Bland Altman graph in Figure 1. The low dispersion of the measurements demonstrates the high repeatability of the LymphoSign signature for the same sample.

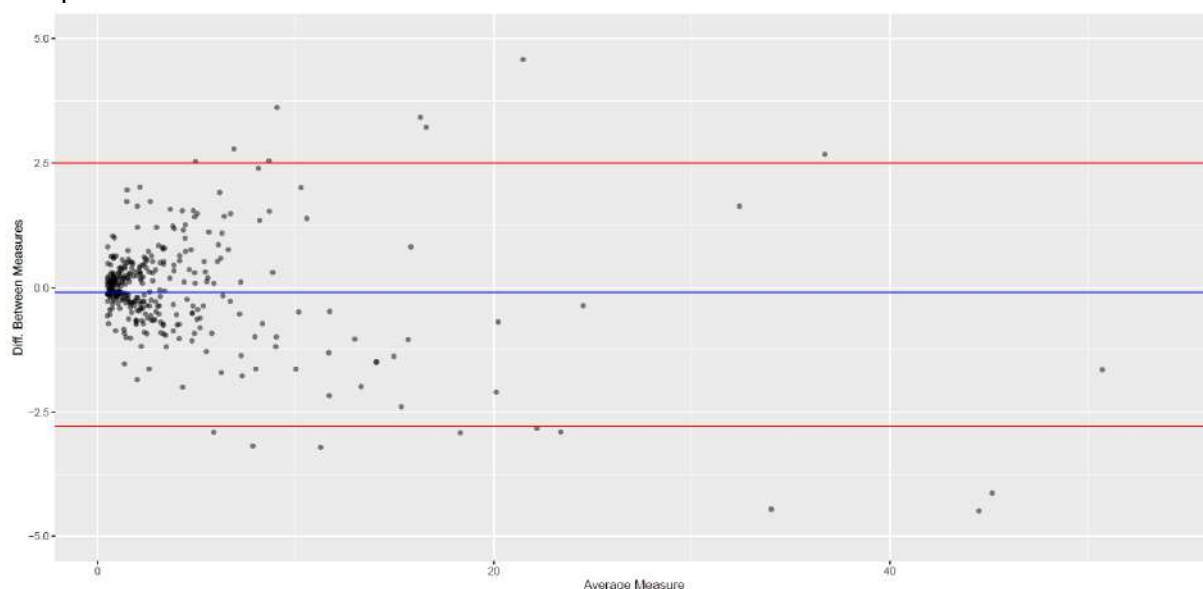
Figure 1: Difference between observed and expected measurements of each test marker for the same sample. The blue line represents the average deviation observed between the average values of each marker and the measured values. The red lines represent the 95% confidence intervals

Interoperability

The Lymphosign test is compatible with Illumina sequencing technology. In order to assess the impact of the nature of the sequencer on the results, 15 samples were analyzed on both MiSeq and NextSeq sequencers. The generated data is perfectly comparable with a Pearson correlation coefficient $r=0.99$ ($p < 2.2e^{-16}$) (Figure 2).

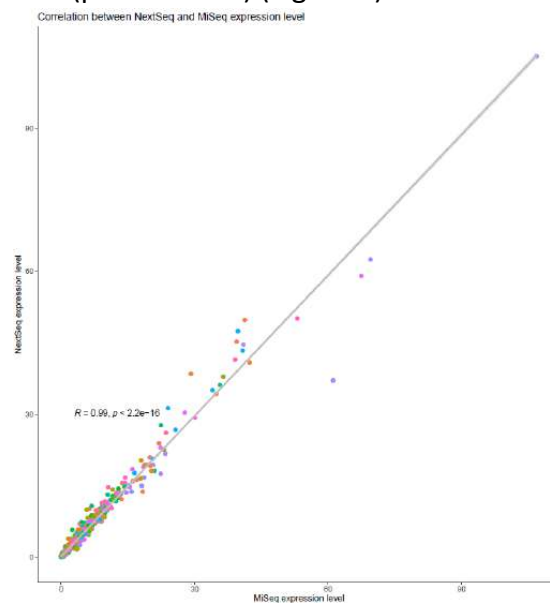
Figure 2: Correlation between marker expression levels of 15 samples sequenced on MiSeq and NextSeq. The coefficient and p -value reported are those of a Pearson linear correlation test.

Reproducibility

The reproducibility between two users was studied according to the expression levels of each of the test markers on 2 samples (Figure 3). The data are perfectly comparable with perfect reproducibility of the measurement of expression of each of the markers (Person: $r \approx 1$, $p < 2.2e^{-16}$).

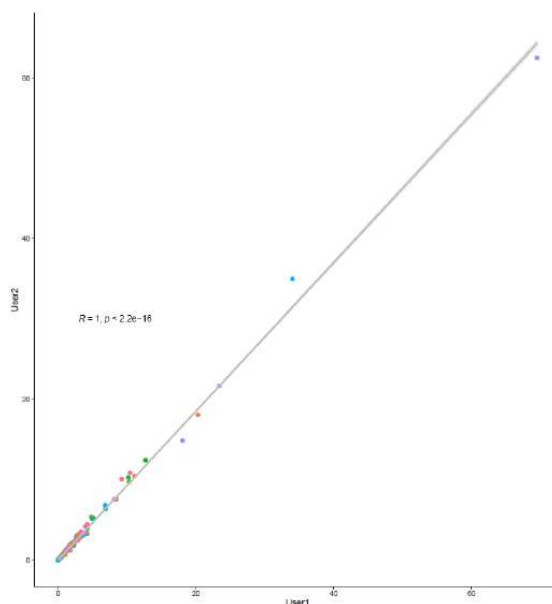
Figure 3: Correlation between marker expression levels between two users. The coefficient and p-value reported are those of a Person linear correlation test.

Bibliography

[Combining gene expression profiling and machine learning to diagnose B-cell non-Hodgkin lymphoma.](#) Bobée V, Drieux F, Marchand V, Sater V, Veresezan L, Picquenot JM, Viailly PJ, Lanic MD, Viennot M, Bohers E, Oberic L, Copie-Bergman C, Molina TJ, Gaulard P, Haioun C, Salles G, Tilly H, Jardin F, Ruminy P. Blood Cancer J. 2020 May 22;10(5):59.

Symbol table

| | |
|--|---|
|  manufacturer |  reagent name |
|  manufacturing date |  temperature limit |
|  shelf life |  see instruction manual |
|  batch code |  CE marking european conformity |
| |  <i>in vitro</i> diagnostic medical device |

Notes

GENEXPATH LymphoSign reagents are protected by intellectual property rights and cannot be modified, reproduced, sold or transferred without the manufacturer's permission. Information in this document is likely to change.

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